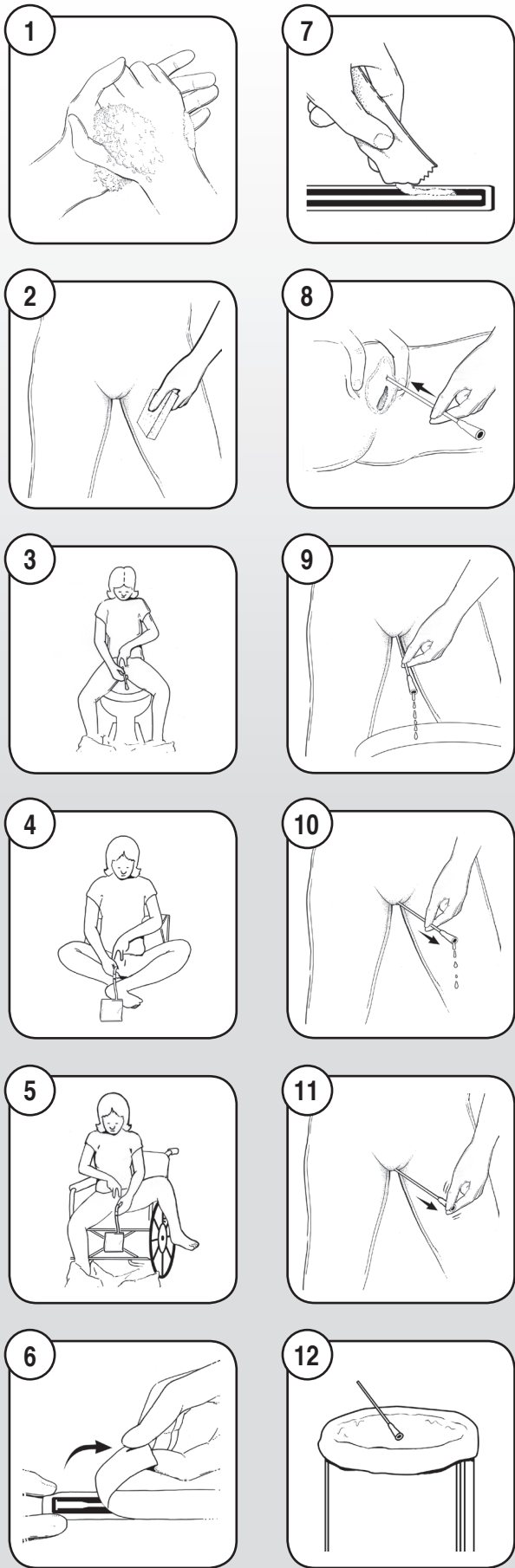
GentleCath™

that's a relief

Please read all instructions before using this device / Por favor lea las instrucciones antes de usar el producto. / Veuillez lire les instructions avant d'utiliser ce dispositif

English

Female Intermittent Urinary Catheters (PVC)

R_x
ONLY

INDICATIONS

Intermittent catheters are indicated for routine short term drainage of the bladder. The catheter is inserted through the urethra.

CONTRAINDICATIONS:

A healthcare professional should assess the need for intermittent catheterisation and the presence of conditions that may increase the risk of catheterisation. This device is contraindicated in the presence of conditions which create unacceptable risk during catheterization.

WARNINGS:

- Selection of an appropriate catheter should be made under the direction of a healthcare professional.
- Do not touch the catheter tip before insertion.
- If there is no urine flow or other complications occur (e.g. pain, difficult insertion or removal, bleeding, signs of urinary tract infection or discharge) please consult a healthcare professional.
- This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection or cross contamination.

CAUTIONS:

- Check the catheter's integrity prior to insertion.
- Select a replacement catheter if required.
- Do not use if package has been previously opened or damaged.
- After incorrect insertion, catheters must be discarded and not re-used. A new catheter should be selected. Used catheters should always be discarded.

INSTRUCTIONS FOR FEMALE USE

NOTE: Healthcare professionals use aseptic technique

1. Wash hands thoroughly with soap and water (*Figure 1*)
2. Clean the genital area from the front to the back, from the inside to the outside, with soap and water. (*Figure 2*)
3. Choose the position that is most comfortable. Move pelvis forward as much as possible to get a better view. Use a mirror to find the opening of the urethra. (*Figure 3, 4, 5*)
4. Peel open the package. (*Figure 6*)
5. Lubricate the catheter with water-soluble lubricant. (*Figure 7*)
6. Take the catheter out from the package.
7. With one hand, spread the labia (the loose tissue on either side of the urethra) apart and lift it gently upwards. The urethra opening should now be visible.
8. With other hand, slowly push the catheter into urethra until the urine starts to flow. (*Figure 8*) Make sure the funnel end is pointing into a container. (*Figure 9*)
9. When the urine starts to flow, push the catheter in 0.4 in (1 cm) - 0.8 in (2 cm) to make sure that it is fully inside the bladder.
10. To make sure bladder is emptied completely, remove the catheter slowly and stop if more urine starts to flow. (*Figure 10*)
11. When the bladder is empty withdraw the catheter slowly. (*Figure 11*)
12. Discard catheter after use. (*Figure 12*)
13. Wash hands. (*Figure 1*)

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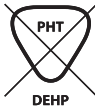
Does not contain phthalate / No contiene ftalato / Absence de phtalates

Do not use if package is damaged / No utilizar si el envase está dañado / Ne pas utiliser si l'emballage est endommagé

Sterilized using ethylene oxide / Esterilizado utilizando óxido de etileno / Stérilisation par oxyde d'éthylène

Does not contain natural rubber latex / No contiene látex de caucho natural / Ne contient pas de latex de caoutchouc naturel

Glossary:

Urethra: Passage which urine is travels through from the bladder to the outside of the body

Labia: Tissue surrounding the urethra and vagina

PVC: PVC is an abbreviated term for Polyvinyl chloride, a synthetic plastic polymer

Intermittent: Occurring occasionally or at regular or irregular intervals

Does not contain natural rubber latex

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© 2018 ConvaTec Inc.
GentleCath is a trade mark of Unomedical AS,
ConvaTec Inc.